

CLEANWND®

Ready to use cloths for easy cleaning and debridement
of the wound and the wound environment.



CLEANWND®

Ready to use cloths for easy cleaning and debridement
of the wound and the wound environment.

WHAT IS CLEANWND®?

CleanWND® is a non-woven, impregnated cloth that acts as an innovative debridement and therapy system for complex wounds. It removes debris and exudate thoroughly and practically painlessly from the wound and eliminates flakes and keratosis from the surrounding skin.



CleanWND® is manufactured using the “needle punch method” and contains short fibers, enabling it to collect wound debris easily during mechanical debridement without leaving any residual lint or fibers in the wound bed or on the surrounding skin.

HOW DOES IT WORK?

The CleanWND® cloths are impregnated with sodium hyaluronate and phospholipid.

Sodium hyaluronate is an important component of the extracellular matrix of the skin and plays a significant role in the metabolism of the dermis, the healing of the wound and tissue repair.

Phospholipids support the cleaning of the skin and replace the phospholipids that have been lost in the damaged tissue of the wound and the surrounding skin.

WHAT ARE THE BENEFITS?

The CleanWND® cloths are already impregnated, so no extra liquids are needed to clean the wound.

Easy to use.

Removes debris from the wound, flakes and keratosis from the surrounding skin.

Effective, fast and simple.

Reduces costs and time.

High acceptance rate by patients, almost painless in use.

Suitable for use on complex wounds.



INSTRUCTIONS FOR USE

Open the packaging where indicated and extract the cloth. Use the cloth to cleanse and debride the wound. The CleanWND® cloth doesn't have a top or bottom side. Use one CleanWND® cloth per wound to perform debridement. The CleanWND® cloth is already impregnated, so no other cleaning fluids are needed to clean the wound. After use, please dispose the cloth as clinical waste.

INDICATIONS

For cleansing/debridement of complex wounds such as;

DIABETIC FOOT ULCERS

VENOUS LEG ULCERS

PRESSURE SORES

COMPLEX WOUNDS / TRAUMA WOUNDS

The product is not intended for use on large wounds..

CLEANWND®

Ready to use cloths for easy cleaning and debridement
of the wound and the wound environment.



COMPOSITION

Sodium hyaluronate
Phospholipid
Aloe Vera Gel

PRODUCT CODE	PRODUCT DESCRIPTION	SIZES	PACKAGING	SALES UNIT
2015A	CLEANWND® debridement cloths	15 x 20 cm	1 cloth/pouch	1 box with 10 cloths



PROTEX HEALTHCARE,
Duwijkstraat 17, 2500 Lier, Belgium
info@protexhealthcare.com, www.protexhealthcare.com