INSTRUCTIONS FOR USE

EN

Name

BAPSCARCARE S silicone sheets.

Product description

BAPSCARCARE S is a proven technology for treating scars (hypertrophic and keloid) and preventing abnormal scar development after surgery or injury. The medical silicone sheets are self-adhesive, thick, washable, transparent and can be used for long periods.

Composition

- Polyurethane film
- Medical-grade silicones

Characteristics

- BAPSCARCARE S silicone sheets ensure that scars become flatter, more flexible and thinner and that complaints (persistent redness, 'pulling' sensations, itching, restriction of joint movement, discomfort and cosmetic concerns) are reduced.
- The silicone sheet creates a semi-occlusive (covering) layer on top of the skin which results in increased hydration of the skin (epidermis). The increased hydration ensures that damaged, scarred skin recovers more quickly and functional problems (complaints) are reduced.

Indications/applications

BAPSCARCARE S silicone sheets are suitable for:

- both old and new scars
- all skin types
- all ages
- the treatment and prevention of abnormal scars (hypertrophic and keloid)
- the treatment of burn scars
- the treatment of surgical scars
- use under clothing and in compression therapy
- the treatment of persistent functional problems caused by the scar (persistent redness, 'pulling' sensations, itching, restriction of joint movement, discomfort and cosmetic concerns)

Instructions for use

- For an optimal result, it is best to start using BAPSCARCARE S silicone therapy immediately after wound closure (after the sutures have been removed and any scabs have disappeared).

- It is recommended to build up BAPSCARCARE S silicone therapy to a maximum of 23 hours a day. The purpose of this build-up is to allow the skin to gradually get used to the silicone.

- For the best possible result the silicone sheets should be worn for a minimum of 12 to a maximum of 23 hours a day.

- If necessary, the BAPSCARCARE S silicone sheet can be cut to size.

- Make sure that the silicone sheet covers the scar and a 2cm-wide area around it.

- To ensure good adhesion and durability of the silicone sheet, it is advisable to remove any hair around the scar location.

- Thoroughly clean and dry the skin before applying the silicone sheet on and around the scar.

- Remove the first part of the protective film of the silicone sheet and carefully apply the self-adhesive silicone sheet, starting at the edge of the scar.

- Remove the second part of the protective film and apply the next section of the silicone sheet to the scar. Depending on the size of the silicone sheet, the above must be repeated until the entire silicone sheet is applied.

- If necessary, the silicone sheet can be fixed in place with an adhesive or elastic bandage.



- The skin may be treated with a hydrating cream after removing the silicone sheet.

Care instructions

- It is necessary to clean the BAPSCARCARE S silicone sheets thoroughly, on a daily basis. This is

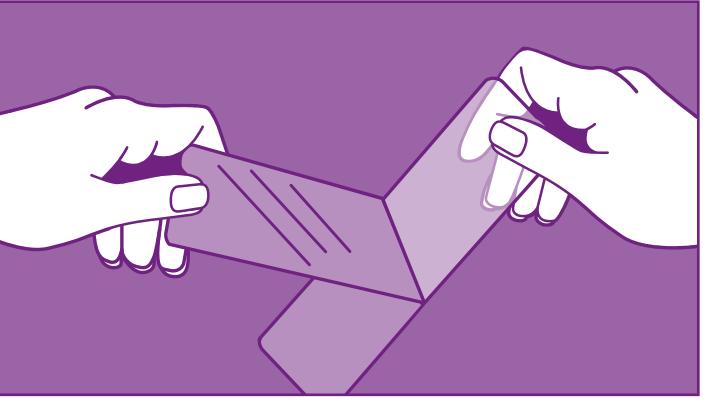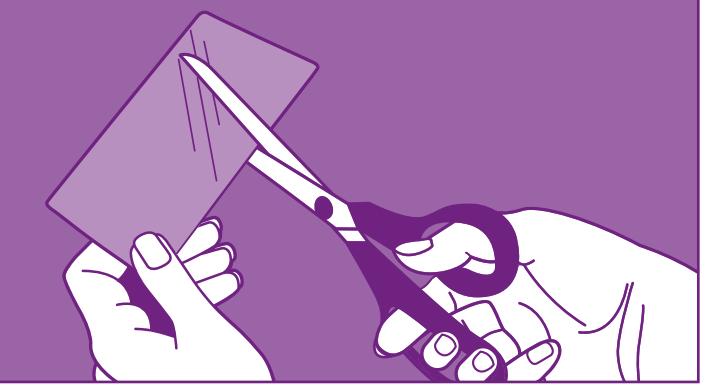Date of last revision of the instructions for use

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