

Product Data Sheet

Article Name:	MEDICOOL Cooling Spray
Article Number:	MEG 600100
PZN	18094349
Product Description / Intended Use:	Immediately relieves pain by quickly cooling the affected areas of the body and is suitable for short-term cold therapy for injuries such as bruises, strains, sprains or muscle injuries. Especially proven in first aid for sports injuries. When sprayed on the skin, a pleasant cooling effect is achieved. Activates blood circulation and is soothing for stressed muscles. The sporty scent has a pleasantly refreshing and invigorating effect. The product may only be used as directed according to the instructions for use.
Product Specifications:	Product can be sprayed from all positions, economical in use, fast cooling effect
Instructions for Use:	Spray the skin areas to be treated evenly and in circles for approx. 5-15 seconds from a distance of approx. 30 cm or at the longest until ice crystals appear on the skin. Use sparingly! There is a risk of local frostbite if used excessively! For external use only! Do not spray into open wounds! Do not spray near the eyes or on mucous membranes! Do not use on infants and children or on persons with sensory disturbances. Side effects are not known when used as instructed.
Safety Instructions:	Container is under pressure. Protect from sunlight and temperatures above 50°C. Do not open by force or burn, even after use. Do not spray on flames or glowing objects. Keep away from sources of ignition - Do not smoke. Keep out of the reach of children. Vapours may cause drowsiness and dizziness. Harmful to aquatic organisms, may cause long-term harmful effects in the aquatic environment. Only dispose of when completely emptied. Do not allow to enter drains.
Ingredients:	Butanes, Propanes, Ethyl Alcohol, Mentha Arvensis
Packaging Unit:	400 ml Can, cs. = 12 Cans
EC Declaration of Conformity:	The products mentioned fulfil the requirements of Regulation (EU) 2017/745. The products are assigned to Class I. The conformity of the products with the above-mentioned directive is confirmed by the CE mark.

