# **PRACTICAL** TIPS





HYIODINE GEL CONSUMPTION

INFECTED WOUNDS 2 days

CLEAN WOUNDS 3 days

WOUND SIZE	RECOMMENDED AMOUNT	
5 x 5 cm	<b>&amp; &amp;</b>	2 ml
7,5 x 7,5 cm	****	5 ml
10 x 10 cm	*****	7 - 8 ml

Hylodine
HEALING THE DEEPEST WOUNDS

AVAILABLE SIZES:

Hyiodine 22g bottle Hyiodine 50g bottle



Contipro a.s. Czech Republic

+420 465 519 530 sales@contipro.com www.hyaluronanhealing.com



HYALURONIC ACID AND IODINE CLEANSES AND DRIVES HEALING

## HYIODINE HEALS

## THE DEEPEST WOUNDS

Due to its gel form, Hyiodine has the perfect approach for reaching and healing the deepest and other hard-to-reach wounds. As has an all-round effect, Hyiodine can be used throughout whole healing process.

ACTIVELY SUPPORTS GRANULATION

NATURALLY MODULATES INFLAMATION

SAFE FOR ALL TISSUES



## FOR A GUARANTEED RESULT

Hyaluronic acid is unique substance in wound healing. It is one of the crucial components of extracellular matrix (ECM), with the ability naturally organise the entire tissue regeneration process. When combined with iodine, it creates ideal healing conditions in even the most complex wounds, including infected ones.

ACTIVE SUBSTANCE	EFFECT	SAFETY
HYALURONIC ACID 1,5%	Initiates and regulates the healing process Supports granulation Helps with angiogenesis	A natural component of ECM No risk of allergic reactions Safely metabolised
IODINE COMPLEX 0,25%	An effective antiseptic against G+/G- bacteria Does not slow the healing process No bacterial resistance	Natural form of iodine (I2) at low concentration No reported allergic reactions Naturally secreted from the body in urine

## GETTING

### THE BEST RESULTS

Hyiodine is most commonly used to heal chronic wounds, where the healing process needs to be "initiated" and accelerated. It is also acclaimed for the excellent aesthetic result it delivers. The newly created tissue is firm and flexible, with significantly reduced scarring.

DEEP DIABETIC ULCERS AND OTHER CHRONIC WOUNDS

NON-HEALING POST-SURGICAL WOUNDS

COMPLEX ACCIDENT-RELATED INJURIES

## APPLICABLE TO

## **EVERY WOUND SHAPE**

Hyiodine is a viscous gel which has wide versatility of application. It is the ideal choice for treating deep wounds with an uneven woundbed. It can also be used in wounds with exposed bone structures, joint capsules or exposed organs.

#### **DEEP** CAVITIES

Use a Hyiodine-saturated gauze to drain deep cavities.



### **NARROW** FISTULAE

Use Hyiodine into narrow fistulas directly from a syringe.



#### **ROLLED-EDGES** WOUNDS

Apply gauze saturated with Hyiodine gel to fill perfectly the underlying areas of wound.



### **CHANNEL** WOUNDS

Apply a drain made from Hyiodine-saturated gauze so that both ends of the drain protrude.

