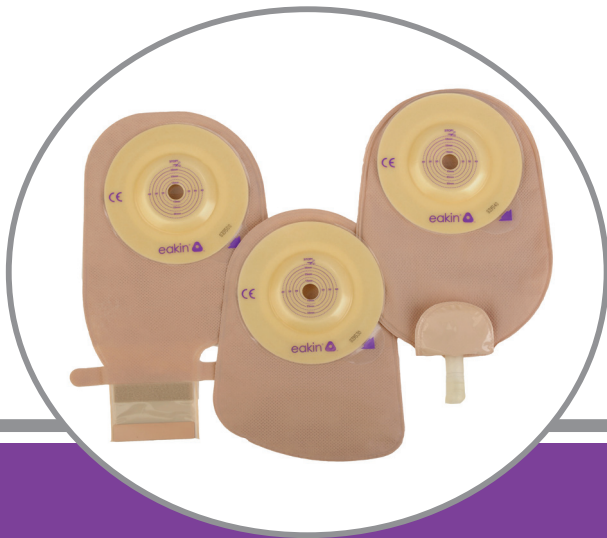
eakin[®]  TM

Cohesive® PASTE
60g

REF 839010

Cohesive® PASTE

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Cohesive
StomaWrap™
10 large oval seals (85mm)

REF 839006

Cohesive
SLIMS®
30 small seals (48mm)

REF 839005

Cohesive®
30 small seals (48mm)

REF 839002

Cohesive®
10 large seals (98mm)

REF 839001

Cohesive®

www.eakincohesiveseals.com

3

FITTING YOUR POUCH

CHOOSING THE CORRECT SIZE*

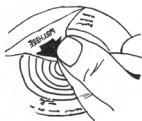
If your stoma is round, then Eakin Pouches pre-cut range will give a good fit. Use the enclosed stoma measuring guide and select a hole which is the same size or slightly larger than your stoma.

*Applies to pre-cut models only

CUTTING THE SKIN PROTECTOR (ON CUT TO FIT POUCHES ONLY)

If your stoma is not round or you wish to cut a particular circle, trace the shape onto the release film and cut with curved scissors.

PREPARING THE SKIN

Ensure the area around the stoma is clean and dry.

APPLYING THE ADHESIVE SKIN PROTECTOR

First remove the release film. Fold back the top of the pouch and locate by placing the lower part of the skin protector below the stoma. Now press the whole skin protector against your skin and smooth it with your fingers (this also helps warm up the skin protector which improves adhesion).

TO REMOVE

Peel the skin protector away from the skin.

FILTER COVER

These can be used on Eakin Pouches and are normally used when bathing or to prevent 'pancaking'. A cover may be applied by lifting up the split fabric and placing the cover over the filter which can be seen as a dark circle.

TO EMPTY DRAINABLE POUCHES

Unfasten the closing strips and unfold the outlet. Empty the contents. To make a larger outlet for emptying and or cleaning, flex the bottom of the pouch and pull the opening tab. Wipe the outlet clean. To close and secure, simply refold the outlet and fasten the closing strips.

TO EMPTY UROSTOMY POUCHES

Pull the outlet closure out of the outlet to allow emptying. To control the flow of urine from the outlet, simply squeeze. To close, simply re-insert the outlet closure.

The following accessories are available to complement your Eakin Pouch.

Cohesive SLIMS® - [REF] 839005

Leakage prevention, comfort and security for all types, shapes and sizes of stoma.

Cohesive® Seals - [REF] 839002

Ideal for extra skin protection, soft convexity, problematic skin or for people with leakage problems.

Large Cohesive® Seals - [REF] 839001

Large Cohesive® Seals cover a larger area of skin around the stoma providing protection over a wide area.

Cohesive® PASTE - [REF] 839010

Cohesive® PASTE is a unique alcohol free paste ideal for filling in deep skin scars and crevices.

If you would like further information on any Eakin product, please contact us via the details on the back cover.

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TG Eakin Limited,