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Instructions for use 1-piece ostomy system Colomate and Ostomate Closed pouches

General information

The protective skin barrier

The skin barriers are made of hydrocolloid, an adhesive made from natural materials such as pectin, gelatine, sodium CMC and synthetic polyisobutylene. No synthetic rubbers (elastomers) or natural rubber latex are added to the skin barriers. The skin barrier has a foam top layer to make the skin barrier pliant, making it more comfortable to wear. The stoma opening in these skin barriers can easily be stretched if it is slightly too small **(G)**.

The pouches

The pouches can have an inspection window, making them easier to apply and allowing inspection of the stoma between changes **(F)**.

The filter

Closed pouches are equipped with a bar filter. To seal the filter off completely, for example when having a bath or when swimming, use the supplied filter stickers. These pouches also have an additional foil layer inside the pouch to protect the filter. This prevents most leakage through the filter.

Intended Use

Closed pouches are intended to be used by ostomates, to collect their (thick) stool. Using this product does not normally require the aid of a medical professional.

Indications

- To be used on intact, healthy skin.
- Can also be used on sensitive or irritated skin.
- Flat skin barriers can be used for stomas above skin level.

Contraindications

There are no contraindications for the use of closed pouches.

Warnings and precautions

- Should irritation occur, discontinue use and consult your healthcare professional.
- Do not use any oil or ointment to clean the skin, the products will not attach.
- Do not use oil or greasy products e.g. bodylotion under the products.
- These products are for single-use only and should not be re-used. Re-use may lead to increased risk of infection and cross contamination. Physical properties of the device may no longer be optimal for intended use.
- Disposal of the pouches should be done through normal household waste.

Instructions

Preparation

Ensure all supplies are available, such as a template, curved scissors, a disposal bag, pen, wet and dry wipes and the (already closed) pouch.

Cutting the skin barrier

- Cut the included cardboard measuring card to the correct shape and size **(A)**.
- Slide the measuring card over the stoma and ensure the opening fits properly and is not too tight around the stoma. No more than 1 to 2 mm of skin must be visible around the stoma **(B)**.
- If the opening fits properly around the stoma, place the measuring card with the printed side on the skin barrier's backing paper. Trace the opening onto the skin barrier **(C)**.
- Cut the drawn shape of the stoma from the back of the skin barrier **(D)**.

- Before applying the pouch, check the fit of the stoma opening through the inspection window of the pouch **(F)**.

Site preparation

- Clean the skin and the stoma with warm water and remove soap remnants. Pat the skin dry.
- Remove the backing paper from the adhesive layer **(E)**. Touch the adhesive layer as little as possible.
- If necessary, stretch the skin barrier slightly **(G)**, to ensure the opening fits around the stoma properly.
- Apply the skin barrier to smooth skin and rub for 30 seconds, particularly around the stoma **(H)**. When applying the skin barrier, it is possible to check through the inspection window whether the adhesive layer is stuck down properly around the stoma **(F)**. The adhesive layer adheres best to slightly damp skin. Pre-warming is not required. The skin barrier will need some time to properly adhere to the skin. The pouches should be worn for no longer than 72 hours before removal.

Removal

Hold the skin and 'peel' the top of the skin barrier downward and off the skin in one slow motion **(I)**. Any thin layer of adhesive that remains on the skin can be covered with the next skin barrier.

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