Pep/Rmt™

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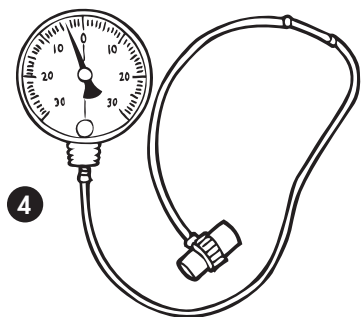
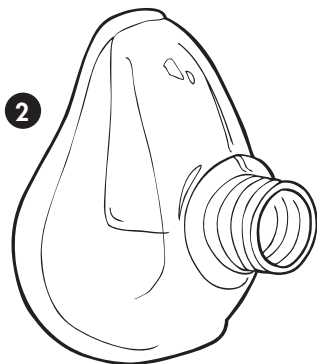
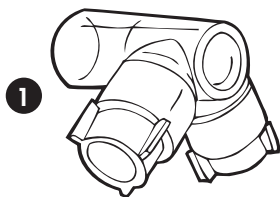
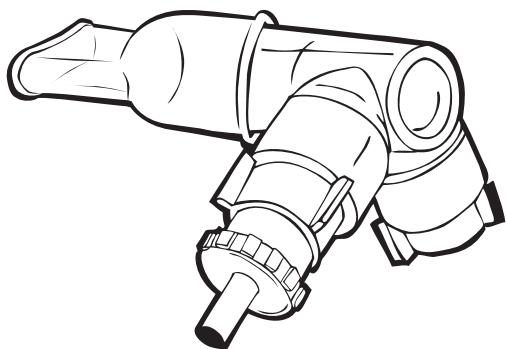
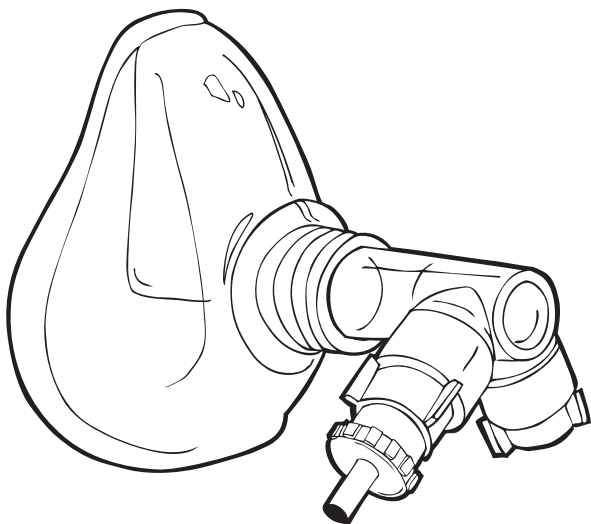
Medioplast AB - Bronsödersgatan 2, SE-213 76 Malmö, Sweden.

Phone: +46 40 671 23 00. www.medioplast.com

Australian Sponsor:
Medioplast Australia Pty Ltd
Suite 102/345 Pacific Highway,
Lindfield, NSW 2070 Australia
Free call 1800 622 492
Website
<https://www.medioplast.com/au>

The Pep/Rmt™ device and treatment should only be initiated against prescription by health care professionals.

Indications

The Pep/Rmt™ set is used to increase reduced lung volumes, decrease high lung volumes, to evacuate sputum and to increase strength and endurance in the inspiratory muscles.

Contraindication

Patients with suspected or known pneumothorax.

Precautions

- Patients with a history of pneumothorax.
- Treatment should only be initiated by specialists in infants or patients with severe obstructive lung disease, since the tidal volume can be in parity with the dead space volume of the mask, there is a risk for raised CO₂ levels in the re-inspired air.
- If the treatment makes you feel dizzy or sick, or if it is unpleasantly strenuous, consult your physiotherapist.

Instructions for use

The Pep/Rmt device consists of;

1. Valve
2. Mouthpiece or face mask
3. Resistor/s
4. Manometer with T-connector*
5. Tracheostomy adapter*

*Accessory, not included. Ordered separately if/when needed.

Parts 1–3 must be fitted together to use the device.

For tracheostomy patients use part 1, 3 and 5.

In order to get feedback from your breathing technique the Pep/Rmt manometer (4) can be mounted between the valve and resistor.

Consult your physiotherapist for instructions regarding breathing technique, treatment program including choice of resistor/s.

If you are using the mask, make sure to keep the mask tight against your face to avoid air leakage.

If you are using the mouthpiece, make sure to seal your lips around the mouthpiece to avoid air leakage.

The manometer (4) can be adjusted with a screwdriver when the plastic plug on the front has been removed.

Cleaning instructions

Table 1 shows the different parts, the material and the proper cleaning, disinfection and sterilizing methods that must be followed, for the parts in question to avoid product material deterioration. Disassemble the parts before cleaning. The T-connector (polypropylene) is a single-use product. If reused, Medioplast cannot guarantee the functionality or the safety of the product. Reuse can cause contamination.

Manual washing or washing machine

Wash the parts in warm water using a detergent suitable for the products' materials. Rinse all parts thoroughly in clean water to remove all detergent residues.

Automatic washing machines designed for medical equipment will normally include program cycles for heat disinfection. Neutral detergent is required to avoid damage to the device or premature material deterioration.

Disinfection, boiling

Use clean water, boil parts for at least 10 minutes.

Disinfection, chemicals

See list of materials in table 1 and follow the instructions on the detergent or chemical disinfectant as to dilution and exposure time. Substances containing phenol should be avoided. After exposing the parts to the chemical disinfectant, rinse thoroughly in clean water to remove all residues.

Autoclaving, 121°C / 134°C

Use standard autoclaving equipment adjusted for max 121°C, or 134°C, whichever is applicable for the product.

Drying and cooling

Leave parts to dry and/or cool completely before reassembling the part.

Inspection of parts

After cleaning, disinfecting and/or sterilizing carefully inspect all parts for damage or excessive wear and replace if necessary. In case of material deterioration, e.g. cracking, the parts should be replaced.

Note:

Some methods may cause discolouration of silicone parts without having an impact on their function. The face masks have an expected lifetime of 1–3 years, depending on usage.

Table 1.

● Applicable ○ Not applicable		Methods					
		Cleaning		Disinfection /sterilising			
		Washing		Disinfection	Autoclaving		
Part	Material	Manual washing	Washing machine Neutral detergent required	Boiling	Chemicals	121°C	134°C
Valve if needed	Polycarbonate / silicone	●	●	●	○	●	●
Resistors if needed	Polypropylene	●	●	●	○	●	○
Manometer if needed*		○	○	○	○	○	○
Mask after each use	Silicone	●	●	●	●	●	●
Tracheostomy adapter if needed	Polycarbonate	●	●	●	○	●	●
Mouthpiece after each use	Polypropylene	●	●	●	○	●	○

* Wipe off with paper or damp cloth with mild soap only.

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