LoFric[®] Sense™

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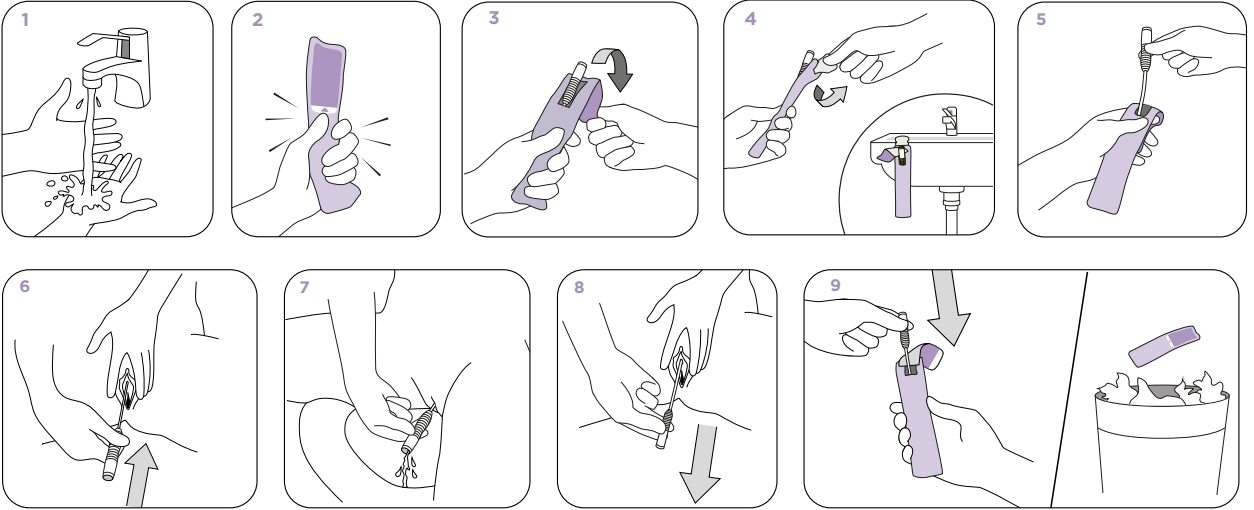
Wellspect[®]
A REAL DIFFERENCE

Manufacturer:
Wellspect HealthCare,

Aminogatan 1, P.O. Box 14, SE-431 21, Mölndal, Sweden
Tel +46 3764000, www.wellspect.com

5500089-C-2021-11-10

UK/CA/AU

INSTRUCTIONS FOR USE:

1. Wash your hands thoroughly with soap and water.
2. Press to release the salt solution and the catheter is ready to use.
3. Pull the tab up to open.
4. Optional: Use the adhesive tab on the reverse side to attach the product to a dry, clean surface.
5. Hold flap in place and take out the catheter.
6. Spread the labia and locate the urethra just above the vaginal opening. With the other hand, insert the catheter slowly into the urethra.
7. When urine begins to flow, insert the catheter slightly more to ensure both eyelets are inside the bladder.
8. When the urine flow stops, slowly withdraw the catheter a little bit. If urine starts to flow again, wait until it has stopped to ensure complete bladder emptying. Then remove the catheter completely.
9. Put the catheter back in the package, the outer packaging doubles as a hygienic and discreet disposal pouch. Dispose appropriately (local regulations may vary).

IMPORTANT INFORMATION

Use only the salt solution included to wet the catheter. Do not place the catheter on paper or cloth as these materials can absorb the wet coating.

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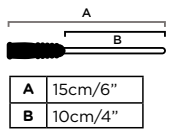
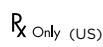
UK/CA/AU

DESCRIPTION OF THE PRODUCT

Single use, hydrophilic urinary catheter with salt solution for activation.

MATERIAL

Hydrophilic coated urinary catheter. Polyolefinbased elastomer (POBE) with PVP coating. Sachet with NaCl and water. Not manufactured with PVC, Phthalates and latex.

INDICATIONS FOR USE

For short and long term bladder management with intermittent urinary catheterization.

PATIENT POPULATION

Female users, adults, adolescents and children.

WARNINGS AND PRECAUTIONS

For single use only.

Once used the surface coating will deteriorate and is no longer sterile.

Reuse may lead to discomfort, urethral damage or infection.

Do not use a product if the sterile packaging is broken or damaged.

Urinary catheterization therapy is associated with an increased risk of urethral bleeding, trauma and/or infection.

LoFric catheters are for prescription use only.

AU: Always consult a health care professional before using LoFric catheters.

Follow instructions and advice from your healthcare professional.

Contact your prescriber if you experience difficulties.

ADVERSE REACTIONS

Common adverse reactions (>1/100) related to catheterization therapy includes urethral damage and urinary tract infection.

If unexpected discomfort, sign of trauma or infection occurs, discontinue use and consult your prescriber.

Any serious adverse reaction occurring when using the catheter should be reported to the manufacturer and your local health authority.

PACKAGE AND STORAGE

Store in their package in a dry place, at room temperature.

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