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EN For decades Van Heek Medical is the supplier for medical products with the highest quality. Products who differentiate themselves in ways of production. All of our products are leading in quality, looks and professionalism. Comfort and user-friendliness is always our highest ambition. Our medical specialists have you in mind while developing products.

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HEKURA

INSTRUCTION FOR USE DAY BAGS FOR MULTIPLE-DAY USE

- NL** Gebruiksaanwijzing meerdaagse dagzakken. Voor gebruik tot maximaal 7 dagen.
- D** Gebrauchsanweisung mehrtägige Urinbeutel. Bis zu 7 Tagen zu verwenden.
- F** Mode d'emploi poches de jour réutilisables. À utiliser pendant 7 jours maximum.
- EN** Instruction for use day bags for multiple-day use. For use up to 7 days.
- ES** Instrucciones de uso bolsas de día de varios días de uso. Para usar hasta 7 días.
- I** Istruzioni per l'uso sacche diurne per più giorni. Per l'uso fi no a 7 giorni.
- N** Bruksanvisning dagposer til flergangsbruk. Til bruk i opptil 7 dager.
- DK** Bruksanvisning dagposer til flergangsbruk. Til bruk i opptil 7 dager.
- S** Användningsinstruktioner dagpåsar för flergångsbruk. För upp till 7 dagars användning.

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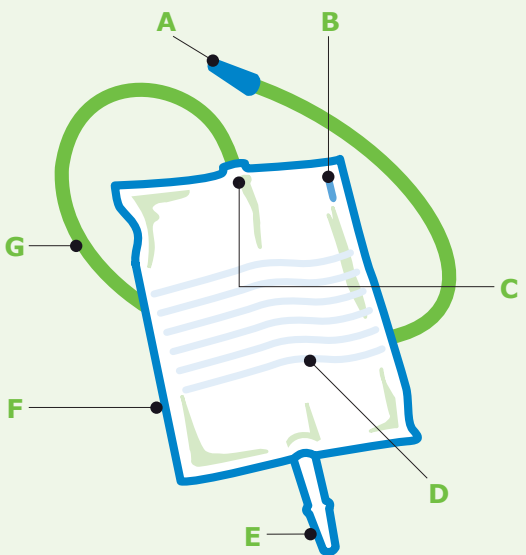
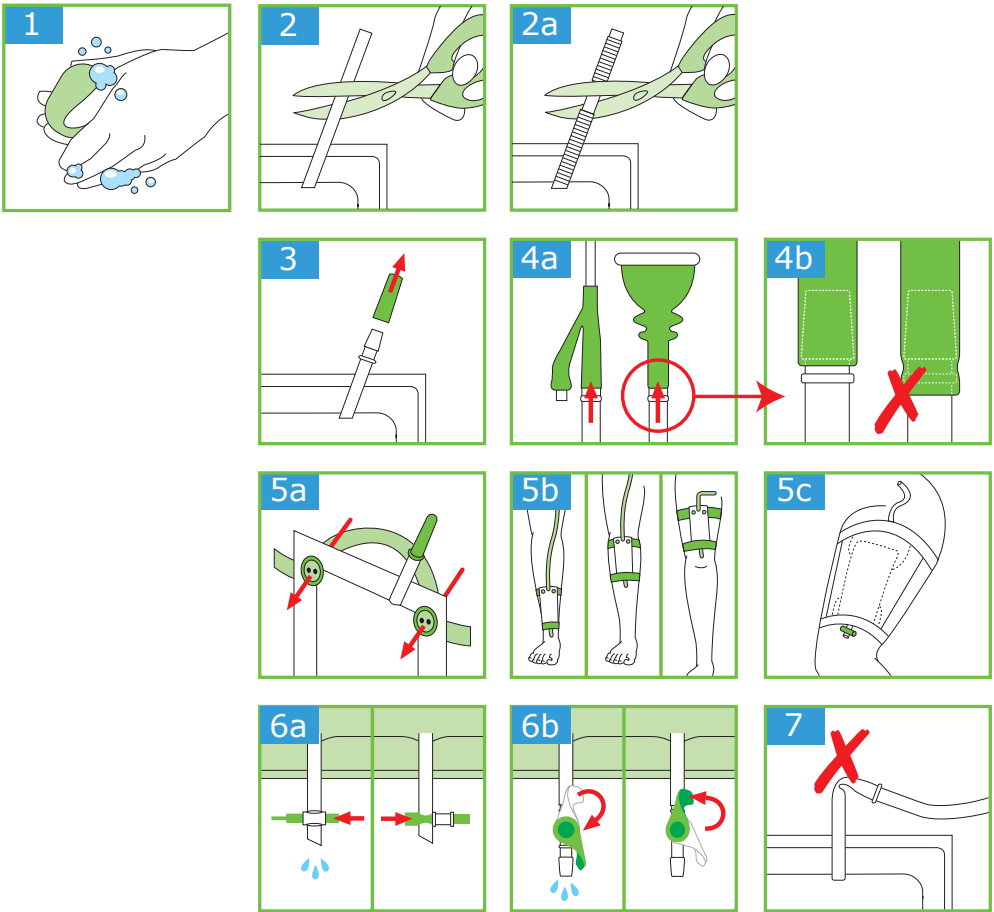
EN **Day bags for multiple-day use**

- 1 Wash your hands thoroughly.
- 2 Optional: Cut the tube to the desired length and firmly secure the connector to the tube of the urine bag.
- 2a Take a urine bag with a ribbed tube and cut into the smooth part. Leave as much of the smooth part as possible to be able to secure the connector properly.
- 3 Remove the green protective cover from the connector and ensure the tap is closed.
- 4 Connect the urine bag to the catheter as far as the ridge.
- 5 Using fixation material, fix the urine bag to the desired part of the leg.
- 6 Open the tap to empty the bag. Clean the end of the tap when the bag is empty.
- 6a Push the cross tap to the other side to empty the urine bag. To close the urine bag, push the cross tap back until you hear a click."
- 6b Turn the lever tap down to empty the urine bag. Turn the lever tap up to close the urine bag.
- 7 Always ensure the tube is not kinked.

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