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EN For decades Van Heek Medical is the supplier for medical products with the highest quality. Products who differentiate themselves in ways of production. All of our products are leading in quality, looks and professionalism. Comfort and user-friendliness is always our highest ambition. Our medical specialists have you in mind while developing products.

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HEKURA

INSTRUCTION FOR USE SINGLE-USE DAY BAGS

- NL** Gebruiksaanwijzing eenmalige dagzakken
- D** Gebrauchsanweisung eintägige Urinbeutel / Tag
- F** Mode d'emploi poches de jour à usage unique
- EN** Instruction for use single-use day bags
- ES** Instrucciones de uso bolsas de día de un solo uso
- I** Istruzioni per l'uso sacche diurne monouso
- N** Bruksanvisning dagposer til engangsbruk
- DK** Bruksanvisning dagposer til engangsbruk
- S** Användningsinstruktioner dagpåsar för engångsbruk

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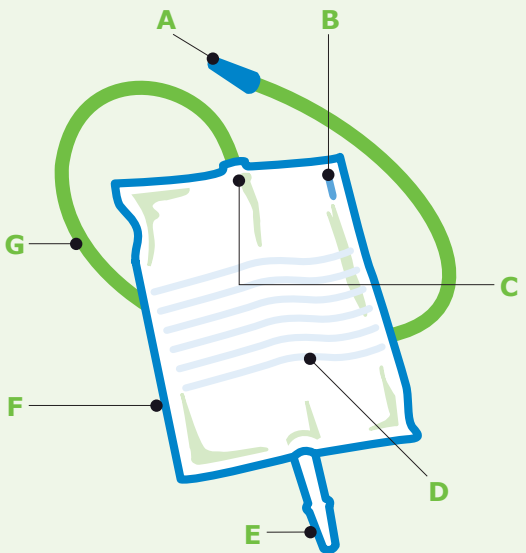
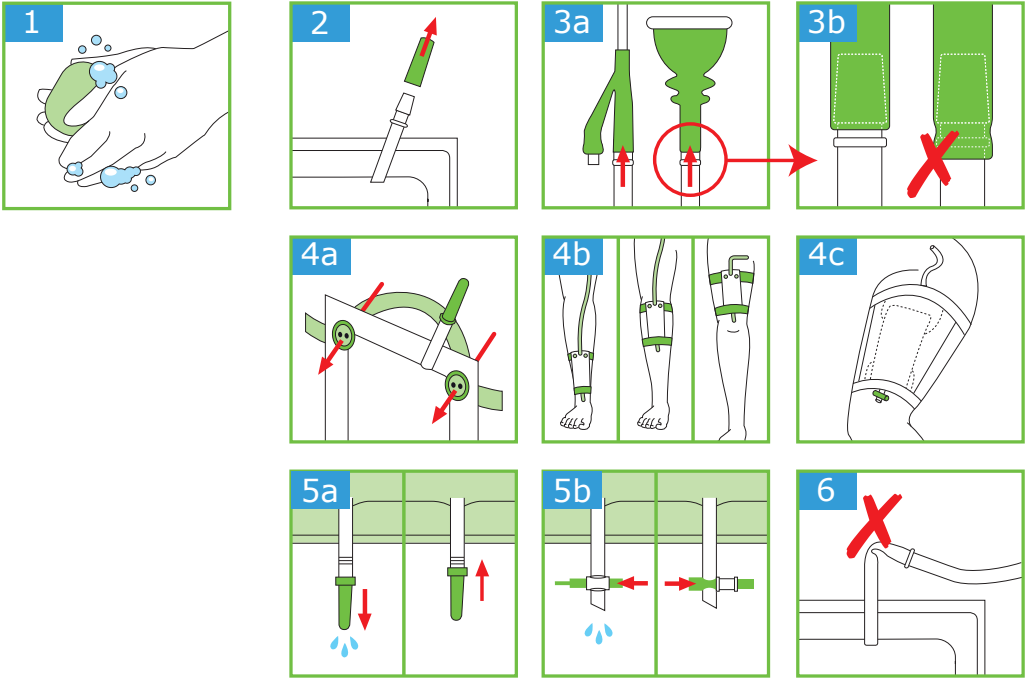
EN Single-use day bags

- 1 Wash your hands thoroughly.
- 2 Remove the green protective cover from the connector and ensure the tap is closed.
- 3 Connect the urine bag to the catheter as far as the ridge.
- 4 Using fixation material, fix the urine bag to the desired part of the leg.
- 5 Open the tap to empty the bag. Clean the end of the tap when the bag is empty.
- 5a Gently pull the pull tap. It will slide down approx. 0.5-1 cm. To close the urine bag, push the pull tap up and firmly secure it.
- 5b Push the cross tap to the other side to empty the urine bag. To close the urine bag, push the cross tap back until you hear a click.
- 6 Always ensure the tube is not kinked.

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- EN**
- A Connector with protective cap
 - B Pre-punched attachment opening
 - C Non-return valve
 - D Calibration is indicative
 - E Drain valve. Tap can be different, read the instructions carefully
 - F Sealing seams
 - G Supply tube. The Hekura urine bag with a short tube contains a lopsided access to prevent hitches in the supply tube.

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