

PRODUCT DATA SHEET

PATSLIDE SLIMLINE PATIENT TRANSFER BOARD

Patient-Transfer Device lets one or two staff members move a dependent patient from one surface to another with very little effort. No lifting, pulling or disruption of sheets. This lightweight patient roller will even transfer the patient up to a height of 75mm (e.g. from a bed to an uneven stretcher).

When you consider the number of patient transfers made daily and the error involved, a PATSLIDE becomes an essential part of hospital equipment in Wards, Casualty, Operating Theatres, Radiology, in fact, anywhere that speed and ease of operation is needed.

PATSLIDE not only makes the job easier and safer, but also reduces the risk of injury to nursing staff when transferring patients. The distress and discomfort that patients can experience when being transferred is also eliminated.

Specifications

- PVC material
- Safe and easy to use
- Static insulated
- Lightweight and resilient
- Radio-lucent
- Highly impact tolerant
- Easy to clean and maintenance free
- Size: 1400W x 480Dx 8mm
- Includes 2 hand holes for easier grip
- Max capacity 120kg
- Colour: White



Order Code	Description
AXPATSLIDE	Patslide Slimline Patient Transfer Board 1400 x 480mm




PATSLIDE SLIMLINE PATIENT TRANSFER BOARD


Patients can be transferred in 3 easy steps

PLEASE NOTE: The PATSLIDE is NOT a lifting device and should not be used as such. The use of the PATSLIDE in any way other than as described is neither recommended nor approved.

- 1 — The drawsheet beneath the recumbent patient is first loosened on both sides and the receiving trolley is butted against the side of the bed.
— The person transferring the patient reaches across and grasps the drawsheet at the patient's shoulder and pelvic level and slightly tilts the patient towards him/herself.
— The receiving person slips the edge of the PATSLIDE Patient Transfer board beneath the patient who is then lowered.

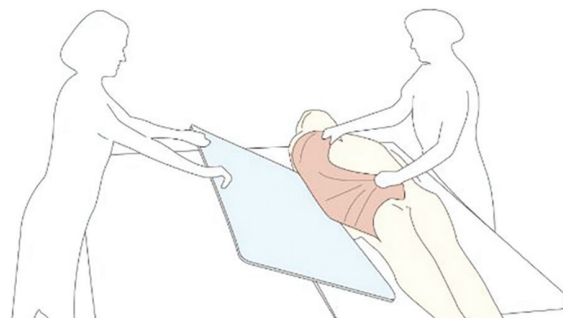

- 2 — The PATSLIDE Patient Transfer board is beneath the drawsheet and is resting partially on both the trolley and the bed, creating a smooth bridge.
— The receiving person grasps the drawsheet at the thoracic and mid-femoral levels and draws the patient towards him/herself as the transferring person assists with a gentle lift.
— The patient slides comfortably and easily to the centre of the trolley.

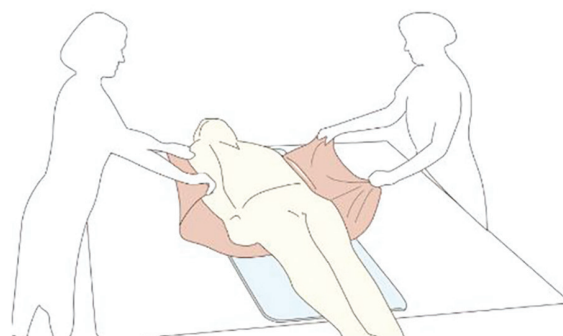

- 3 — The PATSLIDE Patient Transfer board is now gripped by the carry handle and withdrawn.
— The transfer has been completed and with a minimum of patient discomfort.

