INSTRUCTIONS FOR USE

To minimise the risk of cross infection the contents of each container should be used for only one patient. For single use only. Use immediately after opening. Discard after use. Not for injection.

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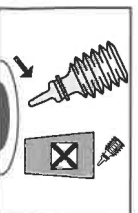
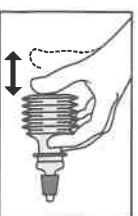
1. Check that the contents are clear and that the flexible container is intact. Do not use if there is any evidence of leakage or interference with the tamper evident feature of the cap.
2. Twist cap in either direction to break the seal and remove cap.
3. Insert the nozzle of the container into the catheter.

4. Hold container neck between first and second fingers and squeeze the bottle by pressing on the base with the thumb to provide a controlled flow into the catheter without the introduction of excess air.

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5. A controlled flushing action can be achieved by gently compressing and releasing the bellows continuously for approximately one minute. Allow the fluid to gently drain back into the container after the agitation period.

6. Discharge the contents of the container down a toilet, sink or equivalent and dispose of the container.

7. Giet de inhoud van de verpakking in een toilet, afvoer of vergelijkbaar systeem en gooi de verpakking weg. NOTE: Studies have suggested that the application of 2 solutions used sequentially will dissolve significantly more encrustation than a single application. If 2 solutions are to be administered, repeat steps 1-7.

NOTE: Studies have suggested that the application of 2 solutions used sequentially will dissolve significantly more encrustation than a single application. If 2 solutions are to be administered, repeat steps 1-7.

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Barb Benelux N.V., Weversdelfhof 10, 3431 HR Nieuwegein, Nederland.

Barb Limited, Forest House, Tilgate Forest Business Park, Brighton Road, Crawley, West Sussex, RH11 9BP, UK

OptiFlo is a registered trademark of C. R. Bard, Inc.

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Patient Information Leaflet

ENG

Please read all of this leaflet carefully before using OptiFlo® R. If you have any questions or are not sure about the use of this irrigation solution then ask your doctor, nurse or pharmacist.

What is OptiFlo® R?

OptiFlo® is a range of different irrigating solutions in a specially designed flexible container. OptiFlo® products are intended to be used by you, your doctor or your nurse in the routine care of your indwelling urinary catheter. Although modern catheters are remarkably free of problems, they still need care if they are to work effectively. The different solutions in the OptiFlo® range have been designed to deal with the occasional problems which are encountered. All OptiFlo® products are supplied sterile.

OptiFlo® R contains a solution called Solution R. This solution can help keep your catheter free of deposits of crystals which might otherwise lead to blockage and infections in your bladder. Solution R can also help in the removal of crystal deposits which have already built up in your catheter. Water alone is not effective in removing these deposits.

OptiFlo® R is provided in two sizes – 50ml and 100ml. Each bottle contains Solution R, which consists of citric acid (6.0% w/v), light magnesium carbonate (2.8% w/v), gluconolactone (0.6% w/v) and disodium edetate (0.01% w/v) in water for injections Ph. Eur.

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OptiFlo is een geregistreerd handelsmerk van C. R. Bard, Inc. 0911/27728
Alliance Pharma S.r.l.