Steve+®

NL

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EN

The new way to put on and remove compression stockings for arm and leg

All over the world millions of people, both young and old, wear therapeutic elastic stockings. Until now, putting on and removing these stockings was no easy task. Steve+® solves this problem.

With Steve+® you can easily put on and remove all types of stockings - with or without assistance. This can be done while sitting or lying down. Steve+® is the perfect aid, wherever you are.

Using Steve+® without assistance

Bending over is no longer required, so even rheumatism or arthritis sufferers can easily use Steve+®. Steve+® is completely demountable, making it easy to take with you.

Using Steve+® with assistance

Steve+® is a convenient solution for medical professionals and caregivers. Any stocking can be put on or removed from the care receiver - with hardly any effort.

A wide range of sizes to avoid wear

Steve+® is available in various sizes to fit all requirements. As a result, the stocking provides more support with minimal wear.

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Armkous aantrekken

- ① Stroop de kous volledig op.

Putting on the arm compression stocking

- ① Roll the stocking up as far as it will go.

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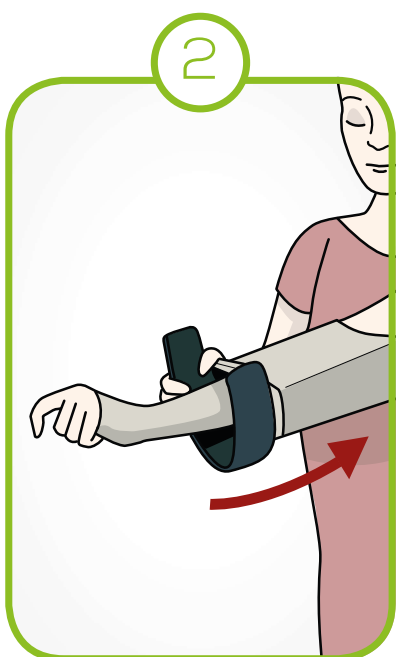
UK

Please note before use:

- Only use Steve+® when seated.
- Do not lean on Steve+®.
- Steve+® is only intended to help with the donning and doffing of medical compression stockings.
- Only use Steve+® as described in the donning and doffing instructions.
- Make sure that the handles **C** have been correctly assembled **A**.
- Make sure that the donning shovel **B** is located firmly in the base **A** before pulling up your stocking.
- When removing the stocking, make sure that the ring **D** is positioned straight in the stocking so that it can be gripped easily by the base **A**.
- Do not store Steve+® near heat sources.

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Indications:

Pain in the back, hands, arms or legs
Rheumatism
Arthritis
Obesity
Problems with bending

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Aluminum Outer Ring
Part code W06-1

Inner Plastic Ring
Partcode W06-2

Inner Plastic Insert Connector
Part code W06-3

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Activate your warranty!

Thank you for your purchase.

To activate your 2 years warranty please visit www.stockingaid.com/warranty

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Washing instructions

Wash STEVE+® with water and soap before rinsing the product thoroughly with water.

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The new way to put on and remove compression stockings for arm and leg.

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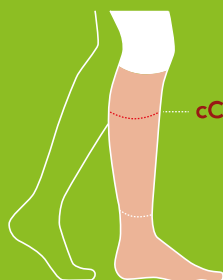
If you want to know more about this globally patented aid,
please visit the website www.stockingaid.com. There is also an instructional video.

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