KALTOSTAT™

Calcium-Sodium Alginate Wound Dressing / Calcium-Natrium Alginaat Wondverband / Calcium-Natriumalginaat Wundverband / Calcium-Natriumalginaat / Medicazione Per Ferite in Alginate Di Calcio E Sodio / Apósito de Alginato de Calcio E Sodio / Tratamiento de Heridas / Apósito de Alginato de Calcio-Sodio para Tratamiento de Feridas / Kalkum-natriumalginaat Sårbehandling / Kalkiumnatriumalginaat-haavavästo / Calcium-Sodium Alginate Sårhåndtag / Kalkium-natrium Alginaat Sårbehandling / Τραυματώδες Απορροφητικό - Ναρπικό / شاشة السليوم سدوم الجيلاتن الجروح

ENGLISH

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION
Kaltostat™ Wound Dressing is a soft, white-to-off-white, sterile non-woven dressing of calcium-sodium alginate fibre. The alginate fibres have wound exudate or saline and are just compressive enough to hold the gel form in place, warm environment to the wound. The gel allows trauma to be removed with little or no damage to newly formed tissue. Exchanging the wound, the dressing may be used for up to 12 months of use. Upon contact with a bleeding wound, Kaltostat promotes haemostasis. This product dressing should be used with a secondary cover dressing.

- INDICATIONS**
Kaltostat is an external wound dressing designed to absorb exudate and promote haemostasis.
Kaltostat Wound Dressing is indicated as an external wound dressing for the management of moderate to heavily exuding wounds such as:
- pressure ulcers, venous stasis ulcers, arterial ulcers,
 - diabetic ulcers and necrotic wounds,
 - donor sites, post surgical incisions, traumatic wounds and superficial lacerations,
 - Kaltostat is also indicated for the local management of bleeding wounds: lacerations, abrasions, nose bleeds, dental extractions, and after surgical wound debridement

CONTRAINDICATIONS
Kaltostat Wound Dressing should not be used on individuals who are sensitive to or who have had an allergic reaction to the dressing or its components. Kaltostat is not indicated for third degree burns or for use as a surgical sponge.

Kaltostat Wound Dressing is not indicated for heavily bleeding wounds.

- PRECAUTIONS AND OBSERVATIONS**
This device is for single-use only and should not be reused. Reuse may lead to increased risk of infection or cross contamination. Physical properties of the device may be altered by heat, moisture, and use.
- Should you observe irritation (redness, inflammation), physical (irritation of skin), hypergranulation (excess tissue formation) or sensitivity (allergic reaction, contact dermatitis) to Kaltostat.
 - Kaltostat Wound Dressing is designed to create a moist environment that assists in wound healing. If a Kaltostat Wound Dressing has initially formed a gel that is allowed to dry out, removal from the wound is difficult. Drying out is generally not a problem with exuding wounds. Kaltostat Wound Dressing should be removed only when the wound is dry. Rinsing the saline may be necessary to remove the gel. If the gel dries out, contact the distal end with saline to rehydrate it; this process can take several hours to return the dried gel to its original state.
 - The dressing may be used on infected wounds only under the care of a healthcare professional.

- Appropriate supportive measures should be taken where indicated (e.g. use of graduated compression bandaging in the management of venous leg ulcers or pressure relief measures in the management of pressure ulcers).
- Colonization of chronic wounds is common and it is not a contraindication to the use of the dressing. The dressing may be used on infected wounds under medical supervision together with appropriate therapy and frequent monitoring of the wound.
- The control of blood clots, as well as appropriate supportive measures, should be provided with diabetic foot ulcers.
- For otolaryngology, a successfully designed of high absorbency is recommended.
- Kaltostat Wound Dressing is not intended for use as a surgical sponge. Kaltostat is not intended to control heavy bleeding. Alternative materials must be considered in those emergency situations where large quantities of blood may be lost.

- DIRECTIONS FOR USE**
1. Wound Site Preparation and Cleansing
Before applying Kaltostat Wound Dressing, cleanse the area with an appropriate wound cleanser.
- Under the care of a healthcare professional, if necessary, before applying Kaltostat Wound Dressing, the wound should be debrided of exudate necrotic tissue and eschar and irrigated with an appropriate non-toxic cleansing solution.

- 2. Dressing Preparation and Application**
a) Kaltostat Wound Dressing should be trimmed to the exact size of the wound avoiding any overlap on surrounding skin.
- b) For moderately and heavily exuding wounds, Kaltostat Wound Dressing should be applied dry to the wound. An appropriate secondary dressing should be used to secure Kaltostat Wound Dressing in place.

- c) When used on lightly exuding wounds, Kaltostat Wound Dressing should be placed on the wound and moistened with sterile normal saline. An appropriate secondary dressing should be used to secure Kaltostat Wound Dressing in place.
- d) Kaltostat Wound Dressing will gel at the wound/bleeding interface and lift away.
- e) Removal from the wound will be accomplished by saturating the dressing with sterile normal saline.
- f) Before applying Kaltostat Wound Dressing, cleanse the area with an appropriate wound cleanser.
- g) As with any dressing, Kaltostat Wound Dressing should be removed from the wound and the wound cleaned at appropriate intervals (e.g. according to Good Clinical Practice, or up to a maximum of seven days).

- 3. Dressing Change and Removal**
a) On moderately and heavily exuding wounds, change the Kaltostat Wound Dressing when mid-absorption of the secondary dressing occurs or whenever fluid is observed.
- b) Removal should be trouble free on heavily exuding wounds. Kaltostat Wound Dressing will gel at the wound/bleeding interface and lift away.
- c) Removal from lightly exuding wounds may be accomplished by saturating the dressing with sterile normal saline.
- d) Before applying Kaltostat Wound Dressing, cleanse the area with an appropriate wound cleanser.

- 4. Bleeding Wounds**
Apply Kaltostat Wound Dressing to the bleeding area to effect haemostasis. Remove carefully when the bleeding has stopped. Apply additional Kaltostat Wound Dressing if necessary.
- Discard any unused portion of the product after treating the wound. Protect from light.
- If further information or guidance is needed, please contact ConvaTec Professional Services.

© 2023 ConvaTec Inc.

0179 indicates a trademark of ConvaTec Inc.

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