GK

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#### DESCRIPTION:

PolyMem polymeric membrane dressings are made of moisture-repelling polyurethane matrix that contains (1) safe wound cleanser, (2) glycerin moisturizer, and (3) superabsorbents. Dressings have a thin polyurethane backing which breathes and helps keep out contamination. Dressings may be used as a primary, secondary, or combined primary and secondary dressing. PolyMem MAX™ dressings have the same polymeric membrane as PolyMem dressings, but MAX is thicker and has higher MVTR to handle more fluid.

Drug-free PolyMem dressings help to:

- effectively manage and heal wounds
- absorb fluid and provide a moist healing environment
- minimize and relieve persistent and procedural wound pain
- reduce edema, bruising, and the spread of inflammation into surrounding undamaged tissues

#### INDICATIONS FOR ALL HUMAN BEINGS:

Over-the-counter, for management of minor wounds, such as:

- Abrasions • Lacerations • Cuts • Burns
- Under the direction of a healthcare professional**, for the management of serious wounds, such as most partial and full thickness wounds, including:
- Pressure ulcers (stages I-IV)
- Venous stasis ulcers
- Acute wounds
- Leg ulcers
- Donor and graft sites
- Skin tears
- Diabetic ulcers
- Dermatologic disorders
- 1st and 2nd degree burns
- Surgical wounds
- Vascular ulcers

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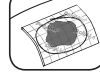
#### DIRECTIONS:

##### Initial Application:

1. Prepare the wound according to protocol or as directed by a physician or other ordering clinician.
2. Select a dressing with the membrane pad ¼" – 2" (0.6 – 5 cm) larger than the wound. The dressing should also cover any inflamed or damaged area surrounding the wound. Dressings may be cut.
3. For dry wounds, moisten dressing slightly prior to application.
4. Apply film side out (so printing is visible).
5. Secure dressings, such as with tape, netting, or gauze.
6. Outline the wound on top of dressing to help determine when to change.

##### Use:

1. A dramatic increase in wound fluid may be observed during the first few days due to the fluid attraction. This is not uncommon and indicates that the dressing is working.
2. Keep the dressing dry and in place when bathing. Change the dressing if it gets wet.

##### Dressing Change:

1. For an exuding wound, fluid will become visible through the top. Change before fluid reaches the wound margin, or when good practice dictates, or after no more than 7 days. Change immediately if fluid reaches the edge of the membrane pad.
2. More frequent changes may be needed due to a compromised immune system, diabetes, infection, hypergranulation, macerated tissue, or when desiring to facilitate the removal of non-viable tissue.
3. Gently remove the dressing. The PolyMem pad will generally not adhere to the wound bed, usually ensuring pain-free dressing changes. Inspect the dressing and wound to assure removal of all material.

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## PolyMem® Pad or Roll Dressing

### The proven multifunctional dressing.

4. If coagulated blood or dried exudate has caused the dressing to adhere to skin or hair, use a small amount of saline or water to slowly gently soften and loosen. Clean the intact skin around the wound.

5. Manage dressing changes according to protocol or as directed by a physician or other ordering clinician. Most cases, when using PolyMem, there is no need to disturb or cleanse a wound during changes unless the wound is infected or contaminated. PolyMem is designed to continuously clean the wound and does not leave residue that needs to be removed.

**Excessive cleaning may injure regenerating tissue and delay wound healing.**

6. Apply a new dressing.

#### Observations during use or change:

- During normal healing, the wound may appear to become larger during the first few dressing changes because the body is naturally removing non-viable tissue. If the wound continues to become larger, consult an expert.
- PolyMem dressings help support new blood vessel formation so it is not uncommon to see blood-stained wound fluid and dressings during changes.
- Be alert for signs and symptoms, such as, increased redness, inflammation, pain, warmth, odor, whitening, fragility, excessive tissue formation, or lack of healing. Consult a professional for problem situations, such as, infection, irritation, sensitivity, allergy, maceration, or hypergranulation.

Any serious incident that has occurred in relation to this device should be reported, by the user and/or patient, to Ferris Mfg. Corp. and the competent authority of the Member State in which the user and/or patient is established.