

Product name/ Trade Name

Elset® - Ref. 3482, 3489 & 993483

Elset® 'S' – Ref. 993486 & 993487

Product description & Indications for use

Type 3A Light Compression Bandage BS7505:1995 to provide compression for the leg following vascular surgery and for general surgical orthopaedic use.

Fibre Content: Viscose, Elastic and Polyester. End Seal - Silicone.

Contraindication(s) – N/A**Warning(s) -**

Caution: This Product Contains Natural Rubber latex Which May Cause Allergic Reactions, including anaphylactic responses.

Precaution(s)

Not recommended for application at full stretch.

Do not use Elset or Elset 'S' if patient/user has known hypersensitivity to the fibre content. If the condition worsens stop use and consult a health care professional.

If soiled with bodily fluids, discard appropriately and replace.

Instructions for use

Elset or Elset 'S' should be applied by a qualified health care professional.

Light Compression Bandage. Apply with a 50% or 67% overlap at mid-stretch using a figure-of-eight bandaging technique.

Special storage conditions and handling conditions

Disposal should be handled according to local environmental procedures.

Store below 25°C (use symbol on ARS)

Other information

If any serious incident has occurred in relation to the use of *Elset* or *Elset 'S'*, it should be reported to Mölnlycke Health Care.