en 2 Piece Drainable Pouch

da Tømlar 2-dels pose

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en INSTRUCTIONS FOR USE

Contents

- Drainable pouch with attached flange
- Ostomy belt tabs (belt not included)
- Integrated Lock 'n Roll™ microseal closure
- Disposal bags

Product Description: Part of a two-piece ostomy pouching system. Designed to be used with a corresponding skin barrier with the same flange size. The pouch is designed to be emptied. The pouch is most suitable for colostomies or ileostomies. Product is made of polymer/copolymer plastics, and nonwoven fabric.

Indications and Intended Use: This product is part of a two-piece ostomy pouching system intended to collect output from a stoma (e.g., colostomy, ileostomy) or skin opening (e.g., wound, fistula).

WARNINGS

Reuse by reprocessing, cleaning, disinfection, and sterilisation (sterilization) may compromise the product. This may result in odour (odor) or leakage.


Notes: Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.


Not made with natural rubber latex. CE mark as required



To close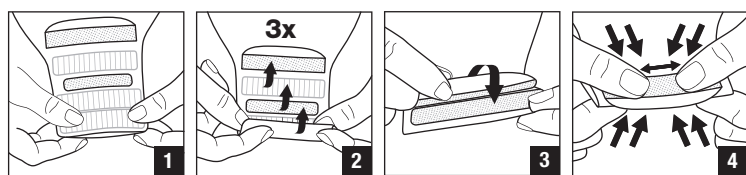


To attach pouch to barrier

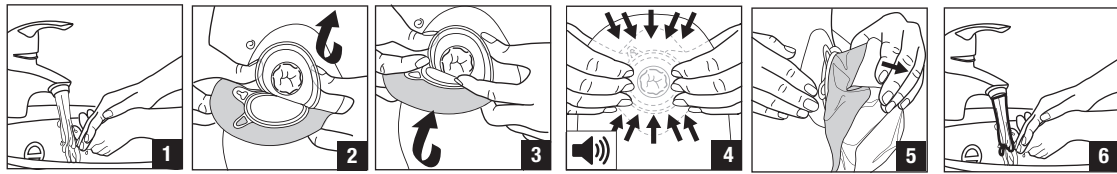To empty

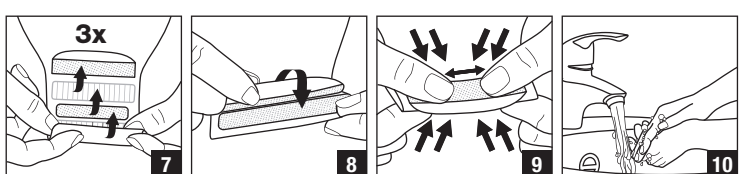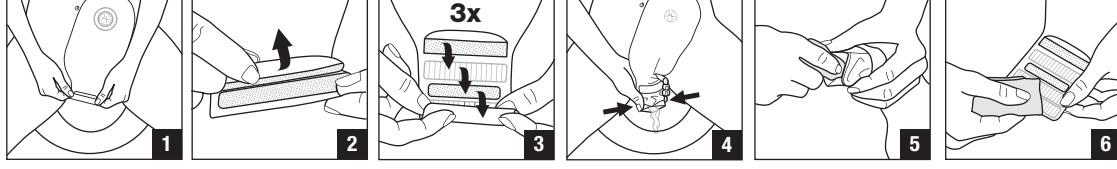To remove and dispose (Please follow all local disposal regulations)

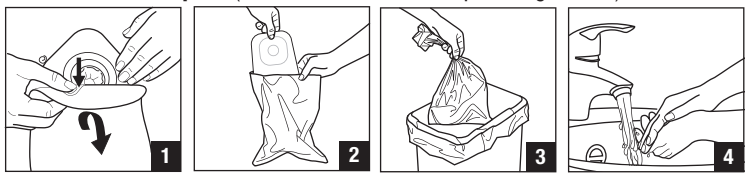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©2019 Hollister Incorporated.