
dansac

www.dansac.com/contact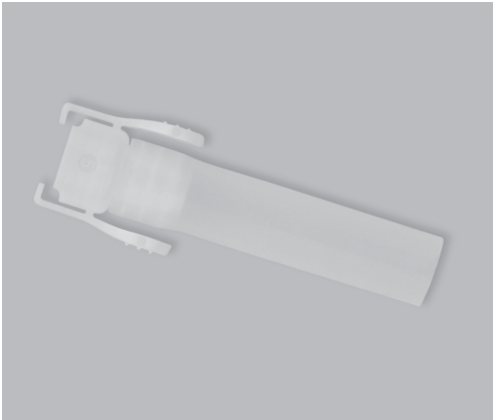

www.dansac.com/symbolglossary

Dansac A/S
Lille Kongevej 304
DK-3480 Fredensborg
+45 48 46 50 00
www.dansac.com

dansac

en • UROSTOMY DRAIN TUBE ADAPTER

Contents: Urostomy drain tube adapter

Product Description: Urostomy drain tube adapter to connect a Dansac urostomy pouch to a night drainage bag or leg bag (not included). Product is made of polymer/copolymer plastics.


Indications and Intended Use: This product is an accessory to an ostomy pouching system that connects a Dansac urostomy pouch to a night drainage bag or leg bag.

Precautions

Single patient - multiple use. Rinse with clean water and leave to dry. Discard when visibly damaged. Sharing the urostomy drain tube adapter with multiple users creates risk of infection.

Notes: Excess mucus from a urostomy stoma may cause the pouch to drain slower than expected. Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.dansac.com/authority or contact local distributor.

INSTRUCTIONS FOR USE

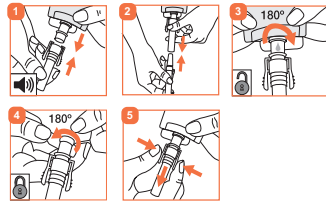To Dispose:

Please follow all local disposal regulations.

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