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Instructions for use 2-piece ostomy system Combimate, Combiplast and Ostomate II Closed pouches

General information

The pouches

The pouches can have an inspection window, making them easier to apply and allowing inspection of the stoma from time to time without removing the pouch (**A**).

The filter (if applicable)

Closed pouches are equipped with a filter. To seal the filter off completely, for example when having a bath or when swimming, use the supplied filter stickers. These pouches also have an additional foil layer inside the pouch to protect the filter. This prevents most leakage through the filter.

The belt connection

The belt connection is made to be used with the EuroTec wide belt with 2 hooks.

Intended use

Closed pouches are intended to be used by ostomates, to collect their (thick) stool. Using this product does not normally require the aid of a medical professional. Closed Pouches are intended to be used in conjunction with compatible skin wafers. Combimate, Combiplast and Ostomate II wafers.

Indications

- To be used for collecting (thick) stool.
- To be used in combination with a 2-piece system wafer.

Contra-indications

There are no contraindications for the use of closed pouches.

Warnings and precautions

All pouches are for single-use only and should not be re-used. Re-use may lead to increased risk of infection and cross contamination. Physical properties of the device may no longer be optimal for intended use. Disposal of the pouches should be done through normal household waste.

Instructions

Preparation

Ensure you have all necessary items at hand.

The application of the pouch

Start at the underside of the flange ring and with two fingers push the pouch onto the flange over the complete circumference. Continue over the left and right side, until you feel a final click. Always check if the pouch is properly fixed, by pulling at the pouch. A pouch should be worn for no longer than 72 hours before removal.

The removal of the pouch

Support the skin wafer and carefully pull the tab on the top side of the flange forward (**B**). Clean the stoma and the area inside the gasket of the flange with lukewarm water before applying a new pouch.

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