BOARD

Pop-On™

SELF-ADHERING MALE EXTERNAL CATHETER

WideBand™

SELF-ADHERING MALE EXTERNAL CATHETER

Spirit™

HYDROCOLLOID ADHESIVE SHEATH

INTENDED USE: The self-adhering male external catheter is used for the drainage of urine. The catheter is applied by the patient or caregiver.

DESCRIPTION / INDICATION: The self-adhering male external catheter is designed for the management of adult male urinary incontinence.

CONTRAINDICATION: Do not use on irritated or compromised skin.

WARNING: Reuse and/or repackaging may create a risk of patient or user infection, compromise the structural integrity and/or essential material and design characteristics of the device, which may lead to device failure, and/or lead to injury or illness of the patient.

PRECAUTION: Do not use if allergic reaction occurs or if patient has known allergies to device components. For good hygiene, change catheter daily. Use of a single device for longer periods than 24 hours may increase the risk of complications. Follow directions to remove, if experiencing swelling, numbness, discomfort, pain, discoloration, or abnormal appearance.

Note: If experiencing problems with use of the device, please consult your healthcare professional for assistance.

DIRECTIONS TO APPLY:

- 1) Verify correct size prior to use.
- 2) Trim pubic hair if necessary.
- 3) Wash penis with mild soap and warm water. Dry thoroughly. Wear time may be reduced if skin is not dry or creasol is used.
- 4) Open package at perforation. Remove catheter from plastic insert, if present.

- 5) Place rolled end over the end of the penis, leaving a small space between the end of the penis and the cone of the catheter.

- 6) Unroll the catheter over penis.
- 7) Gently squeeze the catheter to properly seal adhesive to the skin. If possible, avoid leaving a "collar" around the base of the penis.

IMPORTANT: Wear time may be reduced if adhesive is not properly sealed to the skin.

Connect the catheter to a drainage system. Make sure to check that connections are secure before use.

DIRECTIONS TO REMOVE:

- 1) Ensure drainage bag is empty.
- 2) Disconnect catheter from the drainage system.
- 3) Gently roll the catheter forward and off the penis. If necessary, apply a warm wet compress (such as a wet washcloth) around the catheter to help loosen the adhesive.

DISPOSAL: After removal, dispose by placing the used catheter into a waste container.

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