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Instructions for use of sampling cannulas (Ref. NCE...S NCE...SO) with ENFit, for sampling devices with ENFit for medication bottle (Ref.: NCE20C NCE100C NCE302C) and for milk bottle (Ref.: NCE400C)

1. Indications/Use claims:

Sampling cannulas and sampling devices for the bottle of medication are single use disposable sterile medical devices. They are meant for single sampling of enteral medicinal solutions with an ENFit enteral feeding syringe.
These devices are meant for healthcare professionals.
Target population: Neonatology, Paediatrics, Adult

2. Composition:

- Sampling cannulas (Ref: NCE...S and NCE...SO): PE/PVC, PVC or PU tube, ABS ENfit connector.
- Sampling devices for bottle of medication (Ref: NCE20C NCE100C NCE302C):
NCE20C: PC chamber, ABS ENFit connector and, TPU and PP linked stopper.
NCE100C: TPE and PS adapter, ABS ENFit connector and, PEHD and PEBD linked stopper
NCE302C: screw cap PVC stopper, ABS ENFit connector and, TPU and PP linked stopper
- Milk sampling devices for bottle (Ref.: NCE400C): PP bottle stopper, Europe screw thread, with PE joint and hydrophobic filter, ENFit ABS male tip, TPU link with ENFit PP female cap.

PVC= Polyvinylchloride/ PE = Polyethylene/ ABS = Acrylonitrile Butadiene Styrene/ PP= Polypropylene. /PU : Polyurethane/ HDPE = High-Density Polyethylene / LDPE = Low-Density Polyurethane/ TPU = Thermoplastic Polyurethane

3. Performance/Mechanism of action:

- Devices that enable aspiration and administration of liquids.
- Can be connected to syringes equipped with a standardised ENFit connection (in accordance with standard ISO 80369-3).
- Sampling cannulas are adapted for sampling in ampoules or unbreakable sample vials.
- Sampling devices NCE20C NCE100C NCE302C are intended for medication bottles and are used for inverse sampling.
- The NCE400C sampling device is intended to be screwed onto a feeding bottle containing milk and is used to sample breast milk or infant formula through inverse sampling.

4. Instructions for use:

Requirements for use

These devices are exclusively meant for enteral feeding.

- Check the integrity and cleanliness of the packaging and the device before use; if there are any defects, do not use the device.
- Check the cleanliness of the device.
- Please refer to the hygiene recommendations and procedures in force in the facility for the sampling of enteral solutions.

Positioning of sampling device

Sampling cannulas: Connect an ENFit enteral feeding syringe to the ENFit connector of the sampling cannula. Insert the sampling tube into the container of the solution and take out the desired amount to be administered.

Sampling devices for medication bottles or milk bottle:

- Positioning of the device:

- NCE20C: Tap the capped bottle containing the solution to be extracted.
- NCE100C: Stick the adaptor into the bottle containing the solution to be extracted
- NCE302C: Screw the cap of the bottle containing the solution to be extracted.
- NCE400C: Screw the cap to the bottle to be sampled.
- Connect an ENFit enteral feeding syringe to the ENFit connector of the device.

- Collection : Turn the bottle or the bottle/cap/syringe unit over and extract the desired amount, put down the bottle and take out the syringe. Once the sample has been collected, the bottle can be shut with an ENFit cap.

Administration

To administer the sampled solutions with ENFit feeding syringes, please directly consult the syringes leaflet.

5. Precautions for use and contraindications:

Precautions for use:

- The NCE...S NCE...SO sampling cannulas must be replaced after each use. The NCE...C caps for medication bottles can be maintained on the bottle throughout the period of use for the content specified in the medication leaflet. The NCE400C cap must be replaced after each use or after 24h in the refrigerator.
- After use, the devices cannot be reused and must be disposed of in accordance with the procedures in force.

Contraindications:

- Do not use with pure glyceride solutions (ex.: Acetem, Myvacet, TCM oils).

6. Risks and adverse events:

- There exist no specific risks or adverse events with the devices.

In the event of a serious incident, the user must inform CAIR LGL and the relevant health authority of the country where the incident occurred. The medical devices involved must absolutely be kept so that they can be examined by an expert.

7. Storage/Handling/Disposal:

- Storage: The device must be stored in its original packaging, in a clean, dry area that is away from the light and at a temperature of between +5 and +40°C.
- Shelf life: Refer to the expiry date that appears on the label.
- Waste disposal: To dispose of this device, please refer to the regulation applicable to the disposal of hazardous waste, biological waste or infectious medical waste.

8. Single use:

CAIR LGL did not design this device to be reprocessed, reused or resterilised and, therefore, cannot certify that reprocessing (e.g.: cleaning or sterilisation) will maintain the device's structural integrity and guarantee the safety of the patient and/or user.

9. Glossary of symbols:

Do not reuse. Intended only for single use.	Do not use if the packaging is damaged.	Expiry date.	Always read the instructions for use.	Read the instructions for use relating to warning information	Sterilised with ethylene oxide.	Reference number.	Batch code.	Manufacturer	Date of manufacture.
Medical Device	Do not resterilise								

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