## HEINE Ladebatterien

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## DEUTSCH

## HEINE Rechargeable Batteries

These instructions apply to the following products of the HEINE Rechargeable batteries/battery insert series: NiMH, Li-Ion, Li-Ion L, M3Z, mini2Z, LED S2Z, LED M2Z, K3Z, M3Z 4 NT Li-Ion, M2Z 4 NT Li-Ion, K3Z 4 NT Li-Ion, S2Z 4 NT Li-Ion.

-  Please read and follow these instructions for use of and keep them for future reference.

### Intended Use

The HEINE rechargeable batteries are intended solely for HEINE rechargeable handles.

- **For U.S. only:**  
**Federal law restricts this device to sale by or on the order of a Physician or Practitioner!**

### Warnings and Safety Information

- **CAUTION!** Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

- **NOTE!** This symbol indicates valuable advice. This notes are important, but no related to hazardous situations.

### General Warnings

-  Do not put into fire, do not short-circuit, do not modify, do not sterilise, do not drop, do not expose to severe shocks or to direct sunlight. Dispose of batteries in the event of visible or presumed damage (e.g. leaking liquid, mechanical damage). The batteries can be damaged by liquid. They must not be rinsed in running water or submerged under water. After hygienic reprocessing of the base caps, allow 15 min for the cleaning agent to evaporate before using the handle.

### General Notes

-  The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit [www.heine.com](http://www.heine.com).

Please observe the correct polarity or the instruction/arrow on the rechargeable battery.

Note to the user and/or patient:  
All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

Check the rechargeable batteries regularly. In case of any abnormalities in the operating or recharging times, replace the rechargeable batteries.

If the rechargeable battery has not been recharged for a long period it may discharge automatically. If this occurs a longer charging time will be required when recharging.

### Disposal

-  Dispose at your local collection point.

The appendix contains following tables:

- Technical specification
- Technical specifications of the rechargeable batteries
- Explanation of the used symbols

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