# ARION SlideX

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## Arion SlideX

Extension aid to put on medical compression stockings and tights.  
Covers the leg and reduces friction above the ankle.

### Important information before you start using the product:

- Make sure you're sitting in front of the patient, with the patient's leg resting on your upper leg.
- To avoid additional pressure, place a rolled towel underneath the patient's upper leg.
- Create a double layer by folding the top of the SlideX and connecting the magnets.

### Donning instructions:

- Place the Easy-Slide, Sim-Slide or Magnide (2in1) over the foot.
- At the SlideX, click the magnets together (see C.) and place the SlideX as follows lengthwise on the front of the leg:
  - the long side below (on upper and lower leg), the short side above;
  - wrap the bottom edge around the ankle in a way that the SlideX falls within the Easy-Slide, Sim-Slide or Magnide (2in1);
  - the green loop lies on top of the patient's knee.
- Slide the stocking over the Easy-Slide, Sim-Slide or Magnide (2in1) in the regular way until the heel of the stocking is in the right position.
- Slide the stocking in one seamless move from Easy-Slide, Sim-Slide or Magnide (2in1) over the SlideX further towards the knee up to approximately 2 cm below the bottom of the kneecap.
- The patient can help as soon as the border of the stocking is on the SlideX by pulling the green loop(s) (the XL size has 2 loops) while the caregiver continues sliding up the stocking along the leg.

## SIZING

### For knee-high stockings:

REF	Size	Circumference of the widest part of the calf (1)	
Eo 1520	Medium	≤ 40 cm	≤ 15.7"

### For knee-high and thigh-high stockings:

REF	Size	Circumference of the widest part of the leg (2)	
Eo 1521	Large	≤ 60 cm	≤ 23.6"
Eo 1522	Extra Large	≤ 100 cm	≤ 39.4"

SlideX from Arion Slide Solutions minimizes friction between compression garment and skin above the ankle to the knee (size M) or up to the upper leg (sizes L and XL) thanks to the product's unique sliding properties provided by the special material it is made of and by its coating. It should be used in combination with other Arion application aids such as Easy-Slide, Sim-Slide®, Magnide® and Magnide® 2in1.

### EASY, SAFE AND SIMPLE SOLUTION

- Reduces physical strain for both compression wearers and caregivers
- May contribute to a longer life for the garment and to maintaining the original compression longer by preventing damage from fingers or nails and avoiding overstretching and distortion
- Promotes independence of compression wearers and supports therapy compliance

### COMPOSITION

- Product: Polyester 91%, Polyamide 5%, Silicone 2%, Polyurethane 1%, Magnets 1%
- Package: Chromo carton CC1 (PAP21) 

### CONTRAINDICATIONS

- Do not use the product if you are allergic to one of its component (see composition).
- Open wounds should be covered with an appropriate dressing before using the product.

### ⚠️ WARNINGS

- The SlideX contains several magnets.** Keep this product away from pacemakers and defibrillators. The magnetic field may interfere with the proper operation of this equipment (minimum distance = 30 cm / 11.8").
- Suffocation hazard:** Keep the product away from babies, children, and pets.
- Slipping hazard:** Do not step on the product. Highly slippery material.
- For external use only /** Intended for skin contact. Seek medical attention if irritation develops.

### IMPORTANT

-  Please **read the instructions for use carefully**. If you have any questions, please contact your reseller.
- Please keep the package and the instructions for use during the time of use of the product.**
- All **serious incidents** that occur when using the product **should be reported** to the manufacturer and the competent supervisory authority.
- When using the product it is important to **use the right product size** for the user. Using an incorrect size will prevent the product from working properly, and it may become damaged or wear out too quickly.
- Use the product on a single person.** If the product needs to be used on more than one person, disinfect the product before with a 70% ethanol solution.

### STORAGE

Keep away from direct sunlight or other heat sources. 

### CARE INSTRUCTIONS

- Machine wash at max. 60°C/140°F (colour wash).  
Reduce the amount of laundry, the drum should be filled to a maximum of two-thirds.
- Do not bleach.
- Dry gently in the tumble dryer. Select a gentle program with reduced temperature and duration.
- Do not iron.
- Do not dry clean.
- Disinfection possible with a 70% ethanol solution.

### GUARANTEE

This product is guaranteed for a period of six months from the date of purchase against defects resulting from faulty material or construction. Within the guarantee period, the product will be exchanged free of charge in the case of manufacturing and/or material defects. Any exchange under the guarantee will only be carried out if it can be demonstrated by means of a purchase receipt that the date on which the complaint was lodged falls within the guarantee period. The guarantee does not apply if the defect is due to improper use of the product or use on several patients. To use this service you need to contact your supplier. You are asked to return the product washed and to follow the care instructions.

### RECOMMENDATION FOR USE

 The manufacturer recommends replacing the product six months after the first use if the product is used regularly.

You can write the date of first use on the back of this document.

Watch our **instruction videos** here:  
[uqr.to/SlideXEN](http://uqr.to/SlideXEN) and [uqr.to/SlideXEN2](http://uqr.to/SlideXEN2)  
or scan these QR codes:

More information on [www.slide-solutions.com](http://www.slide-solutions.com)

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Arion Slide Solutions B.V.