

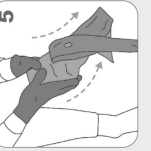
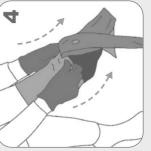
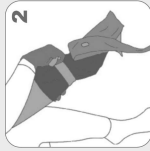
ARION Magnide® 2in1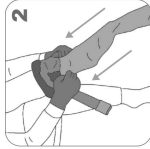

EN	Donning and doffing aid for medical compression stockings with a closed toe	Instructions for use	2
DE	An- und Ausziehhilfe für medizinische Kompressionsstrümpfe mit geschlossener Fußspitze	Gebrauchsanleitung	4
NL	Aan- en uittrekhulp voor medische compressiekousen met gesloten teenstuk	Gebruiksaanwijzing	6
DA	Hjælp til på- og aftagning af kompressionsstrømper med lukkede tæer	Brugsanvisning	8
FR	Aide à l'enfilage et au retrait de bas de compression médicale à pointe fermée	Mode d'emploi	10
NO	På- og avtagningshjelp for medisinske kompresjonsstrømper med lukket tå	Bruksanvisning	12
SV	På- och avtagningshjälpmedel för medicintekniska kompressionsstrumpor med sluten tå	Bruksanvisning	14
FI	Pukemisen ja riisumisen apuväline lääkinällisille kompressiosukille, joissa on suljettu varvasosa	Käyttöohjeet	16
ES	Calzador y descalzador de medias de compresión médica con puntera cerrada	Instrucciones de uso	18
PT	Auxiliar para calçar e descalçar meias de compressão médica de ponteira fechada	Instruções de utilização	20
IT	Dispositivo per togliere e mettere le calze compressive medicali con punta chiusa	Istruzioni per l'uso	22
JP	つま先のある医療用弾性ストッキングの着脱補助具	取扱説明書	24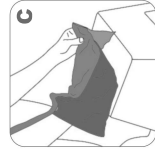

Arion Magnide® 2in1
Donning and doffing aid for medical compression stockings with a closed toe

Important information before you start using the product:

- A. Useful tips:**
 - Wear special non-slip gloves when using the product. This gives you maximum grip on the stocking and reduces the physical strength needed.
 - In case the compression garment has a silicon border, make sure to turn it outwards.
- B.** If you have difficulty to reach your toes, please place your foot on a low chair to avoid bending too much.
- C.** Important! Don't push the foot to the very end of the Magnide® 2in1, leave 2 or 3 fingers space in front of the toes.

On

Donning instructions:

1. Place the Magnide® 2in1 across the foot (ensure that the orange flap near the toes is well folded back, so that the magnets make contact)
2. Pull the stocking over the Magnide® 2in1
3. Slide the stocking over the foot, ensuring that the heel of the stocking sits in the right place
4. Make sure that the light green loop sticks out above the stocking so you can grasp it
5. Pull the light green loop to remove the Magnide® 2in1 from the stocking

Off

Doffing instructions:

1. Slide the Magnide® 2in1 over the stocking and detach the light green loop
2. Fasten the Velcro strap firmly just above the ankle
3. Fold the spare material down over the Velcro strap as far as possible
4. Fold the stocking over the Magnide® 2in1 as far as possible
5. With small movements, starting at the heel, slide the stocking and the Magnide® 2in1 over the foot

Watch our **video instructions** on uqr.to/Mgn2in1EN

Preparation to re-use Magnide® 2in1:

- I Fix the top of the loop to the top of the product by using the magnet
- II Turn over the product and place it on a flat surface
- III Fold both sides with magnets to the center, so that the magnets make contact. The orange stitching is now visible on the outside
- IV Close the orange flap over the product so that the magnets make contact

SIZING

REF	Size	Shoe size EU		Shoe size UK		Shoe size US	
		men	women	men	women	men	women
E01540	Medium	≤ 38	≤ 5 1/2	≤ 6	≤ 7 1/2	≤ 6	≤ 6
E01541	Large	39 - 43	6 - 8	6 1/2 - 9	8 1/2 - 11	6 1/2 - 10	6 1/2 - 10
E01542	Extra large	≥ 44	≥ 9	≥ 9 1/2	≥ 11	≥ 11	≥ 11

XL is also suitable for extra wide feet (foot circumference up to 28 cm / 11")

Magnide® 2in1 from Arion Slide Solutions minimizes friction between compression garment and skin thanks to the product's unique sliding properties provided by the special material it is made of and by its coating. The friction reduction is particularly helpful to slide the garment over the heel area, which is the broadest part of the foot.

EASY, SAFE AND SIMPLE SOLUTION

- Reduces physical strain for both compression wearers and caregivers
- May contribute to a longer life for the garment and to maintaining the original compression longer by preventing damage from fingers or nails and avoiding overstretching and distortion
- Promotes independence of compression wearers and supports therapy compliance

COMPOSITION

- Product: Polyamide 66% Polyester 16%, Polypropylene 10%, Magnets 5%, Polyurethane 2%, Silicone 1%
- Package: Polypropylene (PP)

CONTRAINDICATIONS

- Do not use the product if you are allergic to one of its component (see composition).
- Open wounds should be covered with an appropriate dressing before using the product.

WARNINGS

- **The Magnide® 2in1 contains several magnets.** Keep this product away from pacemakers and defibrillators. The magnetic field may interfere with the proper operation of this equipment (minimum distance = 30 cm / 11.8").
- **Suffocation hazard:** Keep the product away from babies, children, and pets.
- **Slipping hazard:** Do not step on the product. Highly slippery material.
- **For external use only / Intended for skin contact.** Seek medical attention if irritation develops.

IMPORTANT

- Please read the instructions for use carefully. If you have any questions, please contact us regularly.
- Please keep the package and the instructions for use during the time of use of the product.
- All serious incidents that occur when using the product should be reported to the manufacturer and the competent supervisory authority.
- When using the product it is important to use the right product size for the user. Using an incorrect size will prevent the product from working properly, and it may become damaged or wear out too quickly. The correct size is determined by your shoe size.
- Use the product on a single person. If the product needs to be used on more than one person, disinfect the product before with a 70% ethanol solution.

STORAGE

Keep away from direct sunlight or other heat sources.

CARE INSTRUCTIONS

- Wash by hand, at max. 30°C/86°F.
- Do not bleach.
- Do not tumble dry. Dry the product only with a towel.
- Do not iron.
- Do not dry clean.
- Disinfection possible with a 70% ethanol solution.

GUARANTEE

This product is guaranteed for a period of six months from the date of purchase against defects resulting from faulty material or workmanship. Within the guarantee period, the product will be exchanged free of charge in the case of manufacturing and/or material defects. Any exchange under the guarantee will only be carried out if it can be demonstrated by means of a purchase receipt that the date on which the complaint was lodged falls within the guarantee period. The guarantee does not apply if the defect is due to improper use of the product or use on several patients. To use this service you need to contact your supplier. You are asked to return the product washed and to follow the care instructions.

RECOMMENDATION FOR USE



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


The manufacturer recommends replacing the product six months after the first use if the product is used regularly.

You can write the date of first use on the back of this document.

Watch our **instruction video** here: uqr.to/Mgn2in1EN or scan this QR code:

More information on www.slide-solutions.com

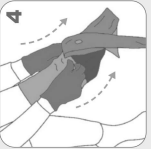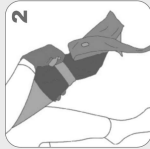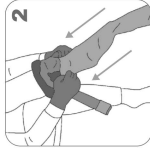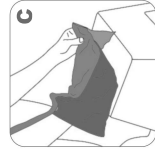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Arion Slide Solutions B.V.